DEBRICHEM[®]
Docnr. INT204.1 V2

CE
1 2 8 2

Warning: Corrosive to skin and eyes, wash off immediately

Warning: Irritant

Warning: Irritant

Barbara Strozziiaan 362
1083 HN, Amsterdam
Netherlands

DEB^X
MEDICAL

(EN) General information

Topical dehydrating agent intended for non-surgical debridement and treatment of non-surgical infected wounds.

Product is intended to be applied by healthcare professionals only.

Product Information

Contains: Methanesulfonic acid, DMSO, fumed Silica, TEOS, Parared. Appearance: red gel like, viscous liquid in vial.

Contraindications

- Ischemic ulcers prior to an effective revascularization.
- Neoplastic Ulcers.
- Underlying abscesses or fasciitis which require lancing.
- Underlying osteomyelitis.
- Intolerance for any of the ingredients.
- Allergies to any of the ingredients.
- The safety and performance of DEBRICHEM[®] has not been substantiated in pediatric subgroups and in pregnant or lactating women. Physicians should consider alternative options for debridement in these patients.
- Underlying cartilage.
- Dry full thickness non-viable tissue (eschar), unless removed pre-debridement.

Warnings

- Wear protective glasses during DEBRICHEM[®] application. Do not exceed 60 seconds application time.
- Wear protective medical gloves made from Nitrile, Polyethylene or Polypropylene.
- Healthcare professional needs to provide adequate protection against exposure of product to the patients’ eyes and mucosal tissue while using the product in close proximity. Drapes must be used to capture liquid spillage.
- Wound needs to be thoroughly cleaned from any substances/ applications previously applied.
- Each vial is for a single application only.
- There are sporadic reports in the literature that acidic topical wound debridement agents can cause delayed wound healing.
- DEBRICHEM[®] should be used in healthcare institutions having the necessary facilities.
- Do not use when package has been opened / seal has been removed from the vial.

Warning: Corrosive to skin and eyes, wash off immediately. Warning: Irritant.

Side effects

- Stinging or burning sensation is expected in patients without sensorial neuropathy, when the product is applied on the wound, but this usually dissipates after a few minutes. Depending on the judgment of the health care professional a topical anaesthetic could be used before DEBRICHEM[®] application.
- Redness, swelling, skin irritation, erythema, first degree burn.

How to apply DEBRICHEM[®]

Preparation

- Put on protective medical gloves, glasses
- Use drapes to cover the wound surroundings. (See warnings)
- Place patient on a treatment table, in a comfortable position such that proper access to the wound can be obtained.
- Wound preparation**
- Remove the existing wound dressing.
- Rinse the wound and periwound with saline, capture spillage with drapes (not shown).
- Clean and dry the wound surface with dry gauze.
- If required, apply topical pain killer, and remove as soon as the anesthetic effect has been reached. (See instructions of the topical pain killer used)
- Shake the vial with the product before opening. Do not use when packaging has been opened before.
- Open vial by removal of the screw cap. (See warnings)

Treatment

- Apply product by pouring generously DEBRICHEM[®] on the wound and leave on for maximum of 60 seconds.
- Uniformly spread DEBRICHEM[®] on the wound with a gloved finger.
- Cover the complete wound bed and periwound.

Cleaning and dressing

- Rinse the wound with ample saline.
- Remove the remaining loosened debris with a dry (sterile) gauze.
- Dry the wound with (sterile) gauze.
- Cover the wound according to standard of care.
- Discard vial with remaining product as indicated in this instruction for use.

General storage and handling

DEBRICHEM[®] should be stored in its protective outer packaging. Keep away from direct sunlight. For storage temperature conditions please refer to the label on the box.

Remove the product from the packaging with careful handling. Prior to opening inspect the package for any damage. Discard the package if the packaging is damaged.

Any unused contents should be discarded by rinsing vial in the sink. Product is for single use only. Do not re-use the product. Disposal: Rinse remaining product in the vial with tap water. Discard the rinsed vial in glass waste and the cap in plastic waste. All chemicals are present in nature and biodegradable.

General safety instructions:

For external use only. Do not use for infusion or injection. Do not swallow. Immediately rinse off with water upon unintended exposure. Do not use on patients when allergic to any of the ingredients Do not use on patients when pregnant or lactating.

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